



If you are having trouble getting the gel to stick to the user, carefully follow these directions:

WARNING:

DO NOT APPLY SKIN TAC TO THE SKIN. REMOVAL MAY CAUSE DISCOMFORT.

Open the packet at the top (you will want to keep the packet to reuse).

Remove the wipe from the packet.

Rub the wipe **ONLY** on the 4 corners of the gel. If too much of the surface of the gel is covered, removal may cause discomfort.

Immediately return the wipe to the packet and seal for future use.

Wait 30 seconds before applying NOIT to the user.

Follow all other instructions for use and care of the adapter and gels from the manual provided.



If the NOIT continues to fall off, you may apply skin tac to the long edges of the gel. Do not cover the surface near the piesos. If the entire surface of the gel is covered with skin tac, removal may cause discomfort to the user, potentially damage the wires inside, and compromise the conductivity of the gel.

If you require more skin tac wipes, you may purchase them easily from online forums like amazon.

Contact NOIT Research™ LLC at www.NOITresearch.org or admin@www.NOITresearch.org for more information.